

A PL1/K1 portable induction loop set

- Free-standing lightweight design, for countertops, desks and tables
- For use in banks, post offices, small meeting rooms, reception desks, open plan offices and ticket booths
- Generates a loop listening field of approx. 1.2 m²
- Comprises:
 - Amplifier with battery and integral microphone
 - Plugtop charger
 - 'AFILS available' sticker
 - Cardboard storage/carry case
- Simple one button operation
- Limited horizontal throw of loop field helps maintain privacy, where confidentiality is essential
- Integral high performance microphone accommodates different operator voice levels
- Remote microphone socket for optional tie/ desk microphone
- Five years expected battery life under normal operating/charging conditions
- Auto shut-off facility (user selectable for 10, 30 or 60 minutes) helps preserve battery life
- Plug-top charger included (amplifier charges fully overnight, but can be used as normal whilst charging)
- Indicators provided for power on, input level, charging required and in progress
- Black plastic






Rear view



Cat. No.	985.40.000
----------	------------

Order qty: 1 set

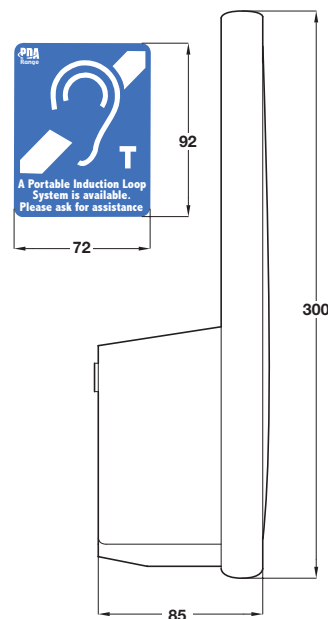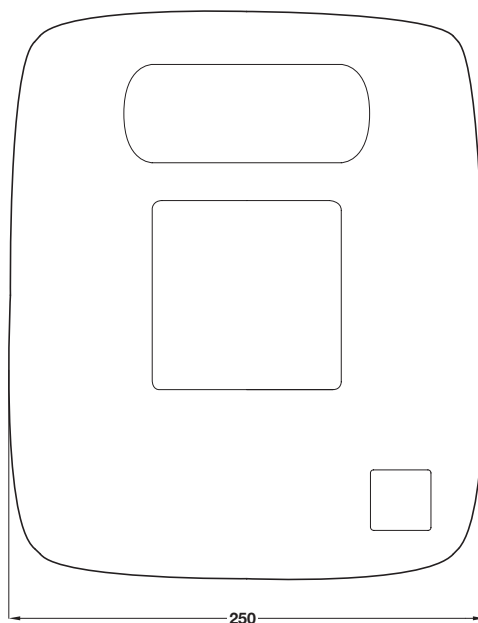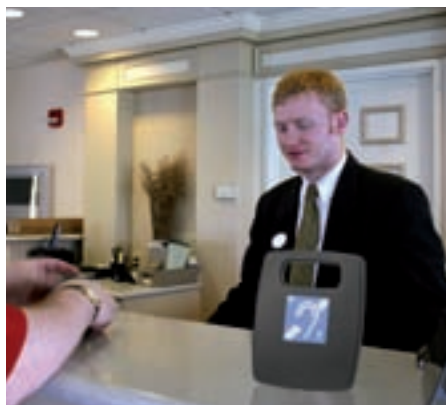

5

Basic operation

- Take the induction loop amplifier out of its box and position it so its front is angled towards the hearing aid user and its back is facing you.
- Ask the hearing aid user to switch their hearing aid to the 'T' position.
- Press the amplifier's on/off button once - the amplifier will beep to confirm it is ready to use.
- Talk to the hearing aid user as normal (you do not need to talk directly into the microphone as it has a wide pick up range).
- Check the amplifier's 'AFILS working' indicators to ensure they illuminate in line with your speech (1 and 2 is normal, 2 and 3 is too loud).

The above instructions assume the PL1 amplifier has sufficient charge. Note that full user instruction are supplied with the amplifier itself